

Soft tissue Massager 01M7-P

Instruction Manual



01M YOUNG ILM
Spine Care System

YOUNG ILM Co., Ltd.

7F, Starwoodplaza, 400, Dunchon-daero, Jungwon-gu, Seongnam-si, Gyeonggi-do, Republic of Korea
Tel: +82-1544-8501, Fax: +82-31-737-0801, Email: 01md@01m.co.kr, www.01m.co.kr

Soft tissue Massager 01M7-P

This instructions for use document covers safe operation of the 01M7-P

Additional information and resources are available upon request or directly from the website, www.01m.co.kr

If the needed information is not found, contact 01M directly at 01md@01m.co.kr

Thank you,

YOUNG IL M Co., Ltd.

7F, Starwoodplaza, 400, Dunchon-daero,
Jungwon-gu, Seongnam-si, Gyeonggi-do,
Republic of Korea
Tel: +82-1544-8501
Fax: +82-31-737-0801
Email: 01md@01m.co.kr , www.01m.co.kr


“Treat and Help the Treated, Give Happiness to the Members and Society”

Table of Contents

1. Structure and Part -----	4-6
2. Specifications -----	7
3. Operating Instructions-----	8-12
4. After Sale Service -----	13

1. Structure and Part

1. Body

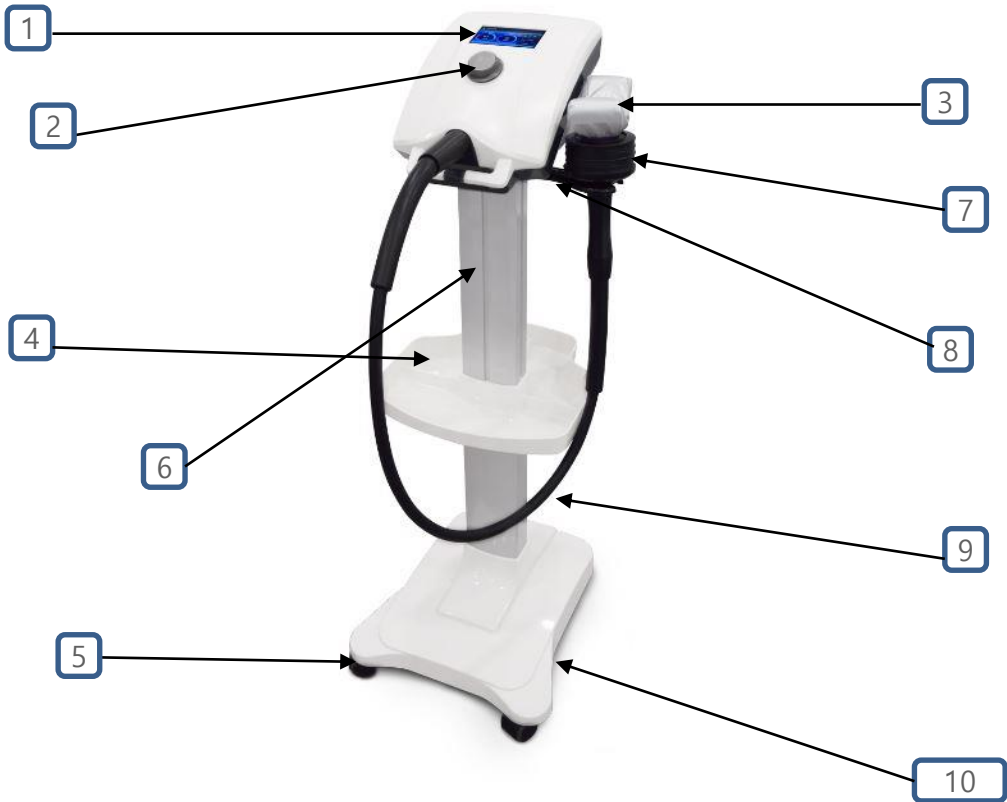

No.	Name	Function
1	LCD Touch Screen	LCD touch screen to operate
2	Dial knob	Time and intensity adjustment
3	Applicator	Transmit vibration to the body
4	Accessory box	Stock the accessories
5	Moving wheels	Easy movement
6	Height stand	Adjust height of the massage machine
7	Hose head	Transfer the rotation power to applicator
8	Head holder	Hang the hose head when not in use
9	Flexible hose	Transfer the rotation to the hose head
10	Bottom cover	Cover the wheels and the bottom structure

1. Structure and Part

2. Components

- Open up box and check the all parts and accessories
(assemble the 01M7-P the same way with old model below)

Main body



8 Accessories



1. Structure and Part

3. Assembling

01M7 to assemble

Take off protection tape on the poll of basement supporter.



①



②



③



④



⑤



Complete

1. Lift up the stand from the lower body.
2. Adjust the tray's height and fix it.
3. The lower body complete
4. Lift up the upper body
5. Connect the upper body with the lower body

2. Specifications



- (1) Dimensions: (W×H) 400*865~1170mm
- (2) Weight : 18.5kg
- (3) Power: 110~230V, 50~60Hz
- (4) Origin: Made in Korea
- (5) Components : upper body, lower body, power cable, 8 accessories(applicators)

Level	CPS	Error range
0	10	(± 20%)
1	25	
2	33	
3	42	
4	50	
5	58	
6	67	
7	70	

3. Operating Instructions

- Refer to instruction manual before operating or servicing system.
- Preparations before use
 - ① The device must be connected to the right voltage and frequency
 - ② Check the status of LCD display and applicators
 - ③ Select the right applicators depending on patients
- Instructions for use
 - ① Turn on the switch located in the rear of body and wait.
(system checking)
 - ② Press the start/stop button to operate
 - ③ Adjust the intensity by dial knob. Operation at high RPM for a long time may lead to overheating or failure
 - ④ Be careful for the cable to not be bent severely
 - ⑤ Do not push down the applicator hard. Please touch the surface of skin smoothly
 - ⑥ Be cautious for the elderly and children with weak bones
 - ⑦ Recommended operation time is 10 minutes and after the operation, please stop the device at least for 5 minutes
 - ⑧ When the device works abnormally, please stop the device and take proper action

3. Operating Instructions

- **Maintenance and Cleaning**
 - After using, please detach the applicators
 - Clean the applicators if necessary
 - Hang the hose head on the holder
 - Unplug when not in use.
 - Do not dismantle or remodel the device.
 - Periodically inspect and maintain the device.
 - When the device is not working, please take proper maintenance and contact supplier(manufacturer).

3. Operating Instructions


Manual Instruction for Use(01M7-P)



First Screen



Unlock by touching

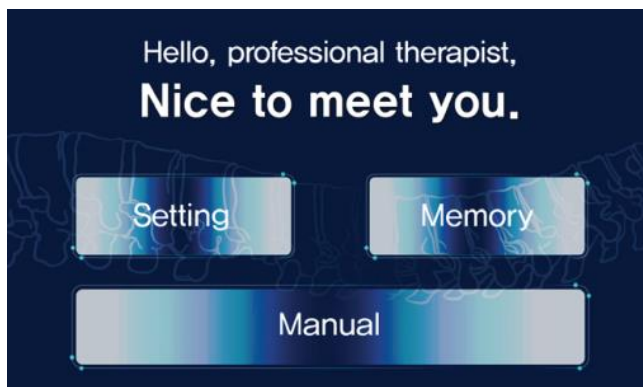


Start your treatment

3. Operating Instructions



Volume and brightness adjustment
(9 level for both)

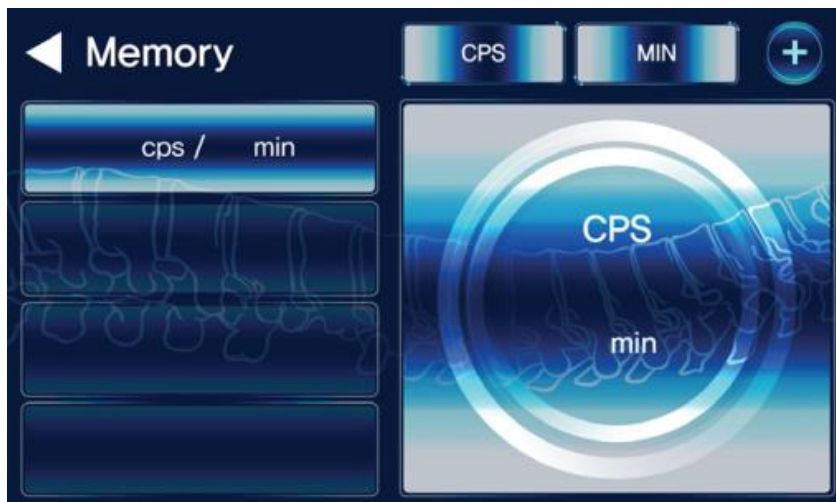

Memory Function
(you can save up to 4)

3. Operating Instructions



Adjust intensity and time by both screen or dial

Level	CPS	Error range
0	10	(± 20%)
1	25	
2	33	
3	42	
4	50	
5	58	
6	67	
7	70	

4. A/S

Product	
Model	
Serial No.	
Date of Purchase	

<Free Warranty Service>

We will guarantee this product for 1 year since the delivery date. If problem occurs even though the product is used properly, the supplier will send or repair the parts at free of cost within warranty term. However, shipping cost might incur on end-user side.

<Exceptive clauses of free warranty service>

The supplier may charge labor and parts cost.

1. When warranty term has expired
2. When user has disassembled or modified
3. Due to any intentional act or gross negligence